





## TECH TIPS

# COLLECTION OF A NASAL SWAB FOR THE **BINAXNOW™ COVID-19 AG CARD** (ANTIGEN TEST)

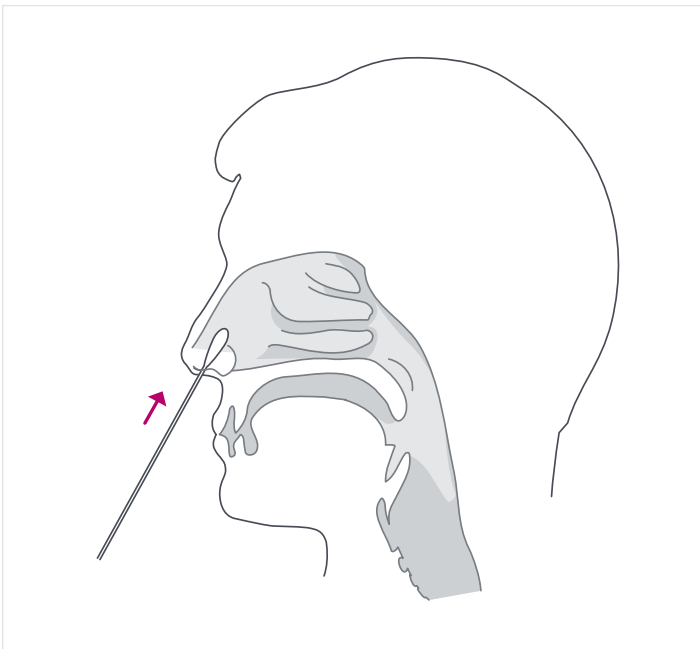

**1** To collect a nasal swab sample, carefully insert the swab into the nostril exhibiting the most visible drainage, or the nostril that is most congested if drainage is not visible.

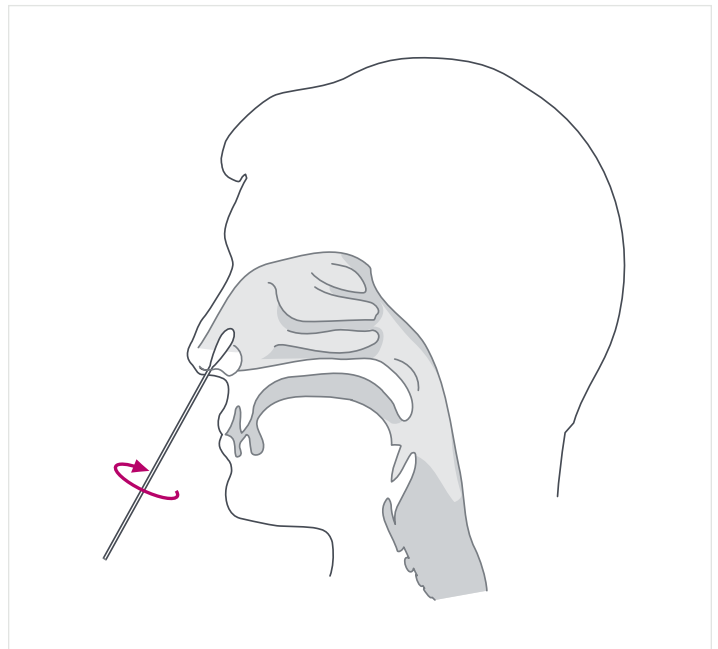

**2** Using gentle rotation, push the swab until resistance is met at the level of the turbinates (less than one inch into the nostril). Rotate the swab 5 times or more against the nasal wall and then slowly remove from the nostril.

**3** Using the same swab, repeat sample collection in the other nostril.

### IMPORTANT REMINDERS

- Please refer to the BinaxNOW COVID-19 Ag Card product insert for full details.
- Use only the swabs provided in the test kit.