

## KN95 Respirator

### Description

The BYD KN95 Respirator is designed to help provide respiratory protection for the user. This product has a filter efficiency of 95% or greater against solid and liquid aerosols free of oil. This product contains no components made from natural rubber latex.

### Intended use

It is intended for respiratory protection against non-oil particles and reduce user exposure to certain airborne particles including those from sanding, sawing, grinding, sweeping, bagging, or from processing minerals, coal, iron ore, flour, and certain other substances.

### Use instructions

- 1 Suitable for protecting the mouth, nose, and chin to airborne particles, and block liquid aerosols, blood, body fluids, and secretions.
- 2 For adult use only
- 3 Do not wash.
- 4 Store in a cool, dry, clean place away from fire and contamination.
- 5 Store between -20°C and 38°C (-4°F and 100°F) with relative humidity below 80%.
- 6 Expiration date: 3 years after production

### 3-Step check before use

- 1 Do not use if the packaging has been opened or damaged.
- 2 Check if the elastic bands are in good condition and are not damaged.
- 3 Check that the metallic strip is not broken.

### Wearing instructions

- 1 Hold the ear loops in both hands with the metallic strip up.
- 2 Put the respirator against your face covering both nose and mouth. Place ear loops around both ears.
- 3 Adjust respirator over nose and under chin.
- 4 Adjust the metallic strip over bridge of nose.
- 5 Make sure the respirator fits properly, covering the nose, mouth and chin.



### Fit test

- 1 Place both hands over the respirator.
- 2 Take a deep breath and hold your breath for a few seconds, making sure the respirator collapses inward as you inhale.
- 3 Then exhale and hold your breath for a few seconds, making sure the respirator bulges outward as you exhale.
- 4 If air leaks, reposition the respirator, tighten the metallic strip and reposition the elastic bands for a better fit. Follow steps 1-3 again until a tight seal has been achieved.
- 5 Respirators that have passed the fitting test are now safe to use.
- 6 Facial hair or other facial features may prevent the respirator from achieving a proper fit. If a tight seal is not achievable, please DO NOT use this product.
- 7 If you experience difficulty breathing, dizziness, or other symptoms, leave the contaminated area immediately.



## KN95 Respirator

### WARNINGS

- 1 The respirator does not eliminate the risk of contracting any disease or infection.
- 2 Improper use may lead to illness and even death.
- 3 Use this product immediately after the package is opened.
- 4 DO NOT sleep while wearing the respirator.
- 5 Avoid hand contact within the interior part of the respirator.
- 6 DO NOT use respirators if they expired.
- 7 For one-time use only. Dispose the respirator according to regulations.
- 8 DO NOT use if the package is damaged.
- 9 Not intended for use in the presence of high intensity heat source or flammable gas.
- 10 This KN95 Respirator is authorized for distribution under FDA Emergency Use Authorization only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of KN95 respirators under section 564 of the Act, 21 U.S.C. § 360bbb-3, unless the authorization is terminated or revoked sooner.

### Main materials

Inner layer	Polypropylene spunbond nonwoven fabric
Middle layer	Polypropylene melt-blown nonwoven
Outer layer	Polypropylene spunbond nonwoven fabric
Ear loops	Polyester/nylon spandex blend
Nose clip	Metal wire with plastic covering



Scan and visit  
BYD Care website



Scan and download  
BYD Care app

### Time use limitation

If the respirator becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and replace and refit the new respirator.

### Important notice

To the extent permitted by law, BYD shall not be liable for any loss or damage including any loss of business, loss of profits, or for any indirect, special, incidental or consequential loss or damage arising from reliance upon any information herein provided by BYD. Nothing in this statement will be deemed to exclude or restrict BYD's liability for death or personal injury arising from its negligence.

Product model: Foldable DG3101

Product meets GB 2626-2006 KN95 standard

